



Q: Can I sleep with the WELDENZ denture in my mouth?

A: Yes, you can sleep with the WELDENZ denture in your mouth. WELDENZ is safe.

Q: Will WELDENZ hurt?

A: In most cases, very little or no preparation of the natural teeth is required for WELDENZ. It is a non-invasive procedure. However, where it involves preparation of the abutment teeth, you may experience a little discomfort associated with the preparation.

Q: Will WELDENZ work for me if I have sensitive teeth?

A: WELDENZ has been used on patients with sensitive teeth. If you have sensitive gums, you might feel some discomfort which is common with other brands of dental restorations. Please consult your dentist who will be able to advise if WELDENZ is suitable for you.

Q: Can I use over-the-counter denture cleaners on my WELDENZ denture?

A: Although it is not necessary, you may use over-the-counter denture cleaners. It will help to preserve your WELDENZ denture.



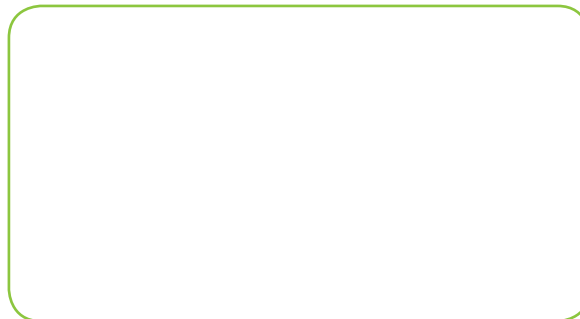
Care for your WELDENZ denture




To prolong the life of your WELDENZ denture, brush it daily with a soft bristle toothbrush as you would brush your natural teeth. There is no need to soak your WELDENZ denture overnight in any special cleaning agent or water. You can safely wear it to sleep.

If a stubborn stain develops that is difficult for you to remove, a trip to your dentist for a routine polishing will remove the stain.

**WELDENZ is the Natural Choice.
Ask your dentist for WELDENZ today!**

Offered by:



 contact@weldenzdental.com
 www.weldenzdental.com
 www.facebook.com/WeldenzDental

© 2015 Weldenz. All Rights Reserved.



WELDENZ[®]
DENTAL RESIN

Giving you

REASONS
to **SMILE**



WELDENZ - The Natural Choice

Natural tooth structure allows better speech, chewing and swallowing. With this in mind, WELDENZ was invented to offer you an affordable and non-invasive treatment option with high aesthetic appeal and to help preserve your natural teeth.

A WELDENZ denture is made from a uniquely formulated thermoplastic resin, comprised primarily of copolymer polypropylene. As a result, a WELDENZ denture is extremely lightweight while delivering superb strength; it also has high resistance to impact and staining.

Most allergies arise from metal used in dentures and bridges. WELDENZ does not contain any metal or bisphenol A (BPA), making it a safe choice for health-conscious patients like you. With its low moisture and non-absorbent properties, it will not cause bad breath which is often associated with wearing dentures. You can safely wear your WELDENZ denture to sleep.

The versatility of WELDENZ makes dental restoration possible with little or no preparation of your natural teeth. Thus, it is possible to preserve your natural tooth structure.

What's more, a WELDENZ denture is easy to care for. No special cleaning solution is required. You don't have to worry about spending extra time and money to maintain your WELDENZ denture.

With WELDENZ, you can enjoy your food and smile confidently at all times.

What do users of WELDENZ dentures have to say



**Dr. Les Latner, DDS, MAGD, AAACD
Los Angeles, California, USA**

*"The WELDENZ material is **strong and lightweight**, making it an ideal material for my patients that require all resin partials and dentures. The appliances that I have delivered to my patients using the WELDENZ Dental Resin are serving them well with hardly any adjustments."*



**Dr. Roger M. Yamashiro, DDS, FACD
Torrance, California, USA**

*"I have placed several WELDENZ partial dentures on my patients and they really like them a lot. Their comments have been: **"it feels very comfortable"**, **"they are less bulky feeling"**, **"it fits better"**, and **"it's very easy to put in and take out"**. The flexibility of the material, it being very hygienic, non-porous and easy to use are tremendous benefits. This is a great material and product that will benefit all practices."*



**Male, 60s, Tokyo, Japan
who received treatment at
Takemoto Dental Clinic**

*"My previous denture fitted me well for a while. However, with receding gum line I started to experience sharp pain when particles of food got stuck between my teeth. I was troubled by the constant pain. The denture adhesive which I bought at drugstores didn't alleviate the pain. Then, I heard about WELDENZ from one of my acquaintances and asked my dentist about WELDENZ. He explained WELDENZ is a very good product, so I decided to try it hoping the current problem would go away. Once I tried, **I felt very comfortable with it. I found the fitting was very good.**"*

Why choose WELDENZ

Flexible and Comfortable



- WELDENZ is **safe**. You can sleep with your denture on.
- WELDENZ is **lightweight**. Being lighter than most leading brands of flexible partials, WELDENZ offers unparalleled comfort.
- WELDENZ has **high resistance to impact**. It does not crack or break easily.
- WELDENZ is **resistant to staining** and **does not discolor**, making it easy to care for.
- WELDENZ is a **non-invasive dental treatment**.



Natural Looking and Compact



Lightweight

Who is suitable for WELDENZ

WELDENZ is ideal for patients who are:

- allergic to metal;
- looking for a more affordable alternative to implants;
- dissatisfied with acrylic dentures;
- not comfortable with other brands of flexible partials;
- seeking non-invasive treatment that can preserve their natural teeth;
- looking for a more durable transitional restoration while waiting for their veneers, crowns or bridges to be prepared;
- looking for a temporary appliance to correct misaligned teeth but do not want the long process of wearing braces. Think "snap on" for instant beautiful teeth!

Do check with your dentist who will be able to advise if WELDENZ is suitable for you.